



# Prestan Professional Manikins

## Replacing Prestan Infant Manikin Limbs

If your infant manikin has lost an arm or leg, below are 2 instruction scenarios for replacing the limb:

### 1. The limb has broken from the socket cavity, but the cap is still with the limb (a normal replacement):


Push on the new leg until you hear a click.




### 2. The limb has detached from its cap (typically unglued) and the cap is still inside the socket cavity:

What you will need: 2 Flat Head Screwdrivers.

Wedge tip of Flat Head Screwdriver between socket and cap as shown.

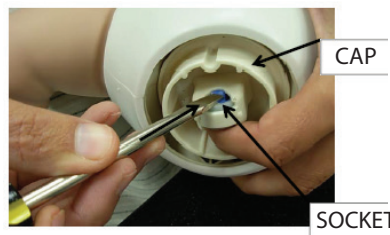

Using 2nd screwdriver, push socket tab inwards while lifting up on cap as shown, then pull out cap.

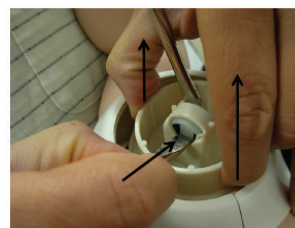

This is what the Leg area will look like once the Cap is removed.



Discard the removed cap along with the damaged limb.

Refer to #1 above to complete the installation.

