

PRESTAN Ultralite® Manikin Frequently Asked Questions



How much does the Ultralite® Manikin weigh?

The PRESTAN Ultralite Manikin units are available in Single (4 pounds), 4-Pack (13 pounds) and 12-Pack (39 pounds). Please check with your PRESTAN Authorized Distributor for availability.

My Ultralite® Manikin gets jammed up when giving compressions.

The Ultralite is specifically designed to promote proper hand position. If you are having problems, please check that your hands are in the proper place and your compression angle is from directly above.

My Ultralite® Manikin slides on the floor when doing compressions.

If you are using your Ultralite Manikin on a bare floor, the PRESTAN branded adult manikin shirt is available with a textured back which will prevent the manikin from sliding on smooth surfaces.

Some of the spokes popped out of the Ultralite® Manikin chest.

Simply unscrew the three screws on the chest plate, remove the top half of the chest plate and reset the spokes, replace the top half of the chest plate.

For more detailed instructions, please see the Ultralite® Manikin Installing/Resetting Chest Plate Help Guide in the PRESTAN Resource Center.

Do Ultralite® Manikins comply with the industry guidelines?

The Ultralight Manikin with feedback complies with the 2019 Guideline changes. Ultralite manikins without feedback can be upgraded to meet industry guidelines by purchasing the Ultralite feedback piston.

Please contact your PRESTAN Authorized Distributor for more information.