

## How to Replace the ‘Rate Monitor’ of the Prestan Professional Manikin

**\* Tools required:**

**Phillips screwdriver**  
**Small Slotted screwdriver**

- Turn the manikin over so that the chest of the manikin is down and the back of the manikin is facing you. Note the 2 parts shown in Figure #1.

**Rate Monitor**

**Wire Cover**

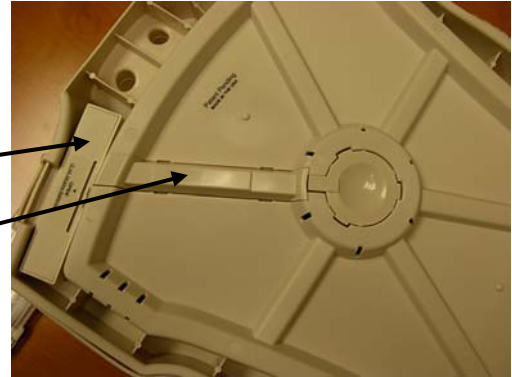


Figure #1

- **1) Remove the Wire Cover** – There are four snap tabs, two on each side of the Wire Cover. Using a small, slotted screwdriver, press in and lift up on two tabs along the same side of the Wire Cover. (fig. #2, #3 & #4)

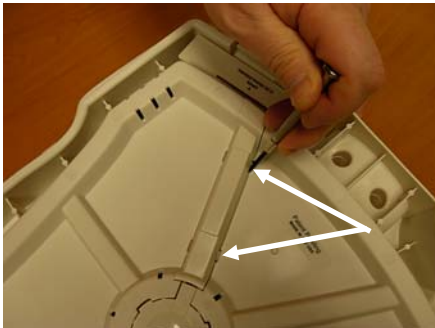


Figure #2

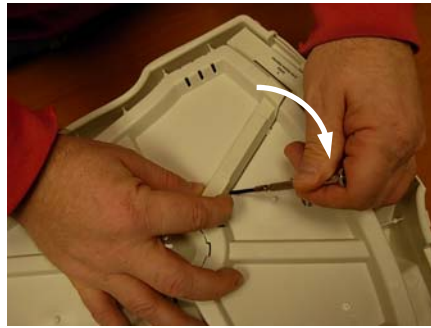


Figure #3



Figure #4

- **2) Disconnect the Wires** – Once the Wire Cover is removed, you will see wires from the Rate Monitor leading to a black Connector in the middle. (fig.#5) Squeeze the small latch on the Monitor side of the Connector (fig.#6) and pull the two halves apart.(fig.#7)



Figure #5

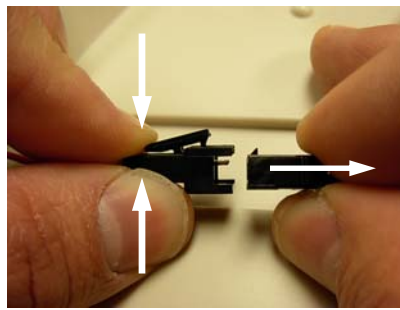


Figure #6

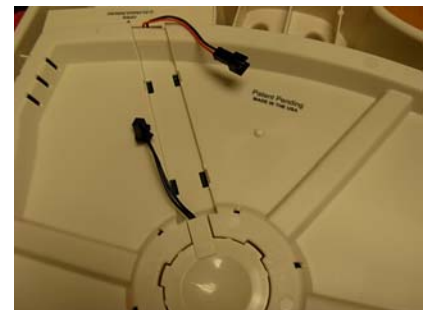


Figure #7



PRESTAN

1-800-434-7989

### **Replacing the 'Rate Monitor' (Continued)**

- **3) Remove the Old Monitor** : Turn the Manikin right side up and remove the Philips screw as shown in figure # 8. Remove the Monitor from the underside of the shoulder as shown in figure # 9.
- **4) Install the New Monitor** : Hold the new Monitor up inside the shoulder and reinstall the screw. (fig.#10)



Figure #8



Figure #9



Figure #10

- **5) Re-connect the Wires and Replace the Wire Cover** – Align the Connectors and press them together. Make sure you hear a click. (fig.# 11) Align wires down the center of channel for the Wire Cover. If needed, make a loop with any excess wire. (fig.#12) Snap the Wire Cover into place by pressing firmly over 4 snaps (fig.#13)

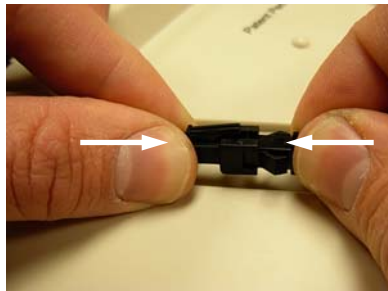


Figure #11

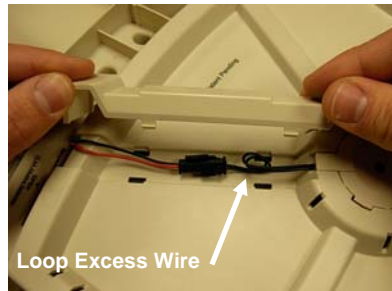


Figure #12



Figure #13