





Prestan Professional Series and Ultralite[®] Manikins


Face-Shield/Lung-Bag Instructions

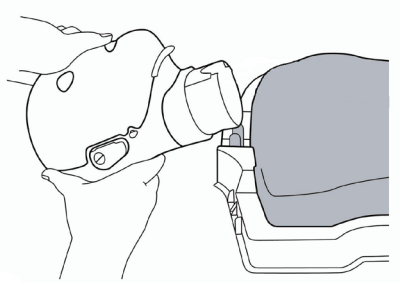


1 Attach head to torso

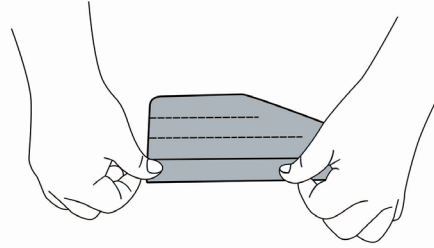


2 Fold lung bag along perforations

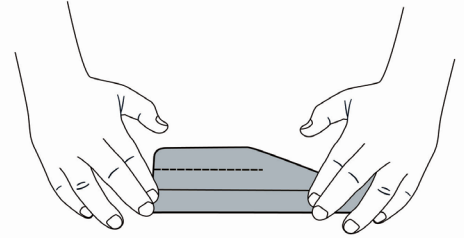


3

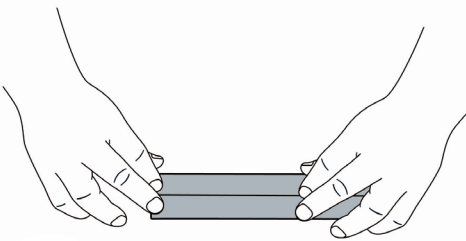


4

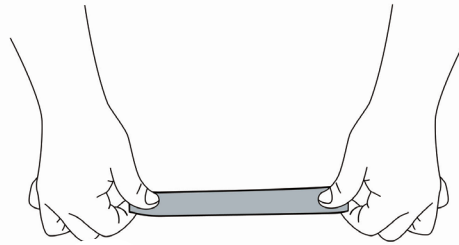


5

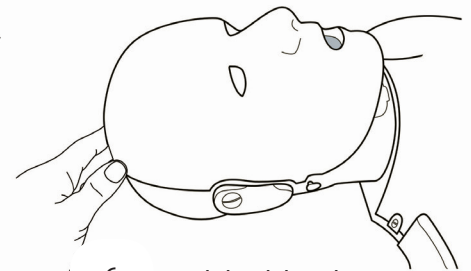


6 Tilt back head

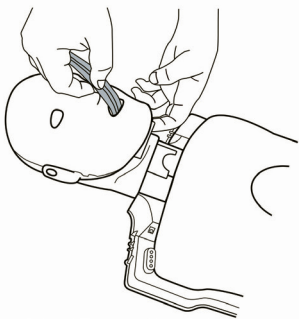


7 Insert folded lung bag into mouth...

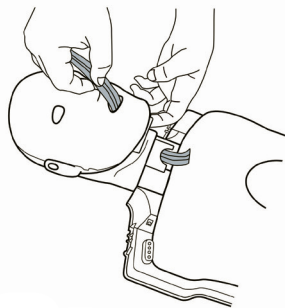


8 ...until it comes out of the neck

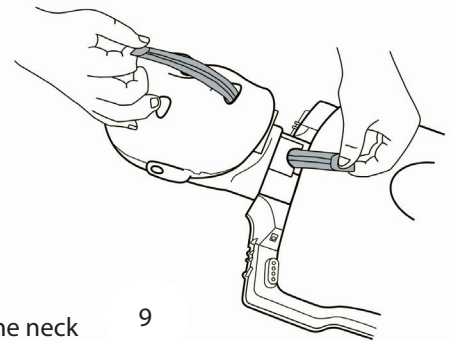

9

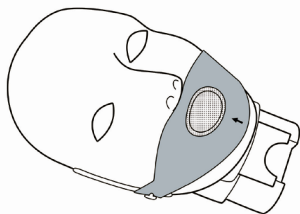

10 Secure face shield to pins under ears

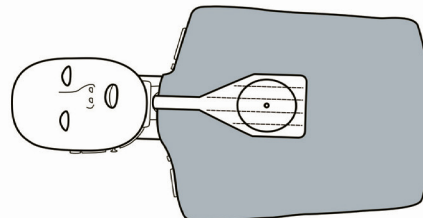

11 Pull lung bag taut to help open mouth hole. Then unfold lung bag onto chest area being sure to align the circle diagram on lung bag with the circle of the chest plate. There will be slack in the neck.

