



Prestan Professional Monitored Manikins comply with the AHA requirement for "instrumented directive feedback".



## Prestan® Professional Manikin CPR Rate Monitor

As recommended by the updated 2015 CPR Guidelines, Prestan has taken its innovative and patented **CPR Rate Monitor** to the next level of quality feedback. When students are practicing CPR compressions on a Prestan Professional Manikin, not only will they see when they have reached and are maintaining the minimum recommended speed of 100 cpm, they will also see when they exceed the upper limit of 120 cpm - the yellow monitor light will flash to let them know!



### How the CPR Rate Monitor Works

1. Begin compressions on the Prestan Professional Manikin
2. When you reach the correct compression depth of 2-2.4", you will hear an audible CLICK to confirm
3. The clicker sends a signal to your CPR Rate Monitor with each compression and the CPR Rate Monitor instantly and continuously responds to your speed with color-coded LED lighting for invaluable feedback!

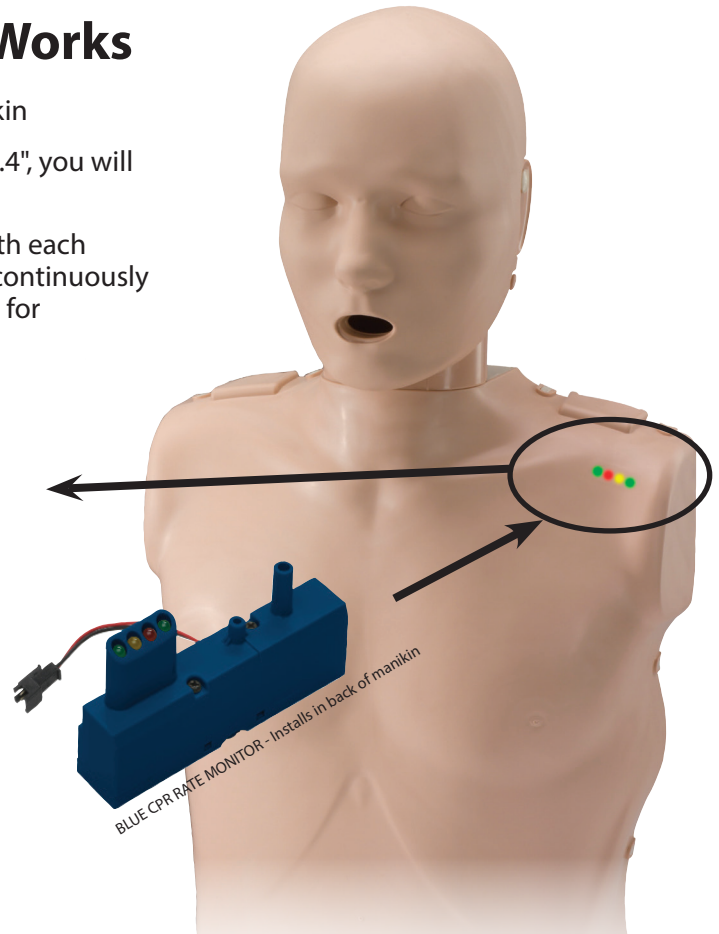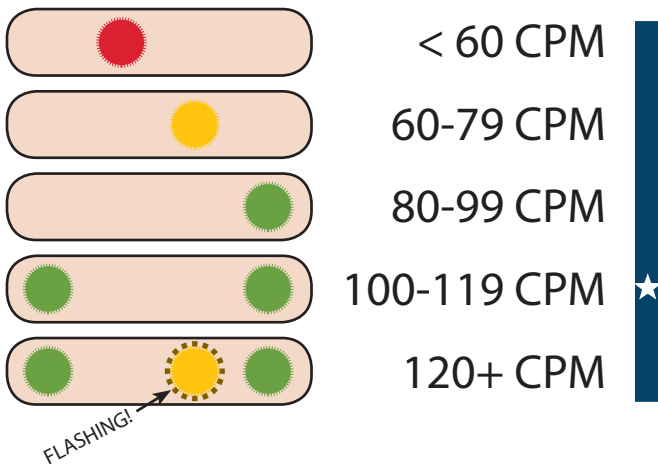

Prestan Professional Manikins may be purchased with or without the CPR Rate Monitor. The CPR Rate Monitor is also available for separate purchase to upgrade existing manikins or easily install into a Prestan Professional Manikin originally purchased without a monitor.

Visit our website at [www.prestanproducts.com](http://www.prestanproducts.com)

701 Beta Drive, Suite 3 | Mayfield Village, OH 44143 | 800-434-7989