

尺寸:250 x 166 mm



Instruction Manual

Contact Information:

WNL Products
275 Hopping Brook Road Holliston,
MA 01746
Toll Free: (800)884-9629
www.wnlproducts.com

Warranty:
One year parts and labor
(excludes battery)

Date: 2018/9/28
No.: WNL-120F-A
Version: C

Product Introduction:

The NEW Practi-CRdM® Compression Rate and Depth Monitor is designed to measure compression rate and depth when practicing CPR on a manikin. Worn on the students' wrist, the CRdM will provide audible "beeps" when compression depth is not within the range of 5-6cm (2"-2.3"). A beep at one time indicates insufficient depth (less than 5cm/2"), and two beeps at one time indicates excessive depth (more than 6cm/2.3"). A diode illuminates red when compressions are below 100 cpm or above 120 cpm. When compression rate is between 100 and 120 cpm the diode illuminates green, allowing users to self-check their timing.

Features:

1. Power on

Touch the screen for 3 seconds to turn it on, touch for another 3 seconds to turn it off.

2. Dual-colored indicator light

The indicator light will flash green when the compression rate is between 100 and 120 cpm. A diode illuminates red when compressions are below 100 cpm or above 120 cpm.

3. Buzzer prompt

The unit will provide audible "beeps" when compression depth is not within the range of 5~6cm (2"~2.3"). A beep at one time indicates insufficient depth (less than 5cm/2"), and two beeps at one time indicates excessive depth (more than 6cm/2.3").

4. Charging

Connect the magnetic charging points of the charging cable to the back of the unit, and then connect the USB port of the charging cable to the USB port of your computer, or the USB power adapter.

The indicator lights red when it is in charge, and it will turn green when it is fully charged.



Components:



Compression Rate and Depth Monitor




Magnetic USB charging cable

尺寸:250 x 166 mm



Storage case

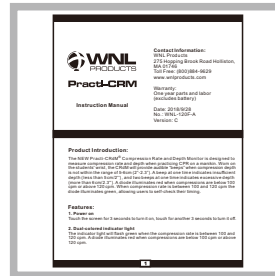

Instruction manual

Operation Instructions:

1. Take out the unit from the storage case and wear it on the wrist.



2. Turn on the unit by touching the screen for 3 seconds, and start CPR.

Hand Placement:

- Place heel of one hand in center of the chest
- Interlock fingers of the other hand
- Lean forward so your shoulders are over your hands
- Press straight down with elbow straight.



Specification:

Model No.: WLCRdM

Name: Compression Rate and Depth Monitor

Power Supply: DC3.7V, rechargeable lithium battery

Static Current: 30μA Max

Working Current: 25mA Max

No.	Contents	Quantity	Unit
1	Compression Rate and Depth Monitor	1	pc
2	Charging Cable	1	pc
3	Instruction Manual	1	pc
4	Storage Case	1	pc