



# Prestan Professional Manikins




## Replacing the Prestan Infant Rate Monitor and/or Batteries

To replace the Rate Monitor on the Infant Manikin, you will need a small Flat Head Screwdriver.

Remove the Monitor Cover (located in the diaper area of the infant manikin) by gently inserting the edge of the flathead screwdriver into one of the tabs of the monitor cover. The cover will pop out easily.

Remove the old Rate Monitor by lifting it straight out of the cavity.

Insert the new Rate Monitor into the cavity and press the cover back onto the diaper area. The lights may begin to blink, but can be shut off by pushing the On/Off Button.

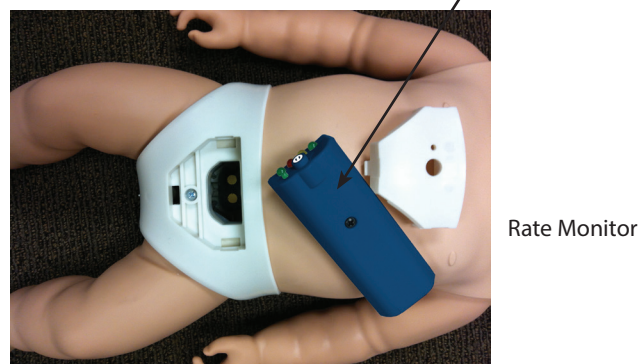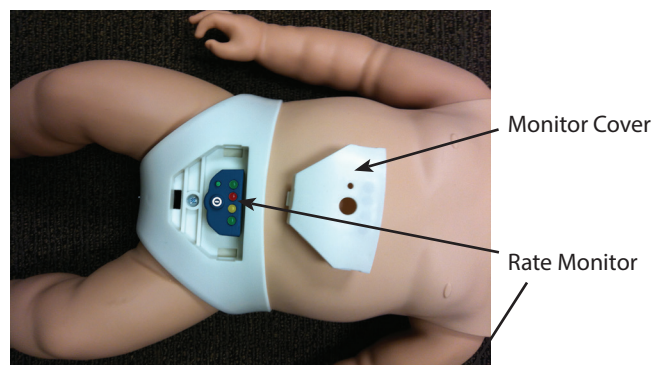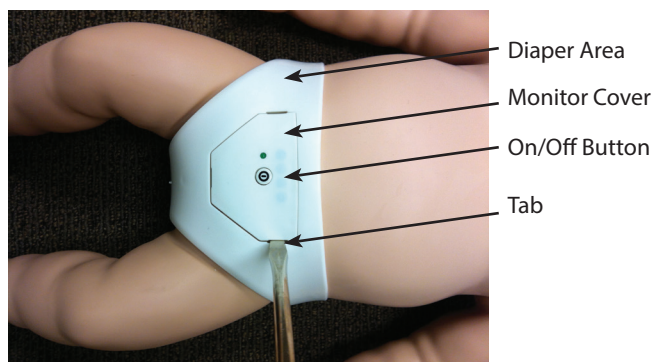

**Battery Replacement:** If all monitor lights are on and remain on, this is usually an indication that the batteries are about to die.

To replace the batteries in the Infant Rate Monitor, remove the battery cover from the back of the monitor and replace batteries with 2 new AA batteries.

