


## PRESTAN Series 2000 Manikin:

### How to Power On/Remove From Demo Mode

To start the PRESTAN Professional Adult Series 2000 Manikin, a one (1) second long compression is required, at which time you will hear a beep and see the LED lights illuminate according to the table below. This will signal that the manikin is ON and ready for use.

**PRESTAN Series 2000 Manikin LED Sequence Table**

	Connected Mode	Disconnected Mode
Sleep Mode/No Power	○ ○ ○ ○	
Smart Device Connected Mode <small>(Cycles in sequence left to right)</small>	● ● ● ●	n/a
1 <sup>st</sup> Full Compression, Not Connected Mode <small>(Wake mode/LED Test Mode)</small>	n/a	● ● ● ●
CPM* < 60	○ ● ○ ○	
CPM* 60 – 79	○ ○ ● ○	
CPM* 80 – 99	● ○ ○ ○	
CPM* 100 - 120	● ○ ○ ●	
CPM* > 120	● ○ ● ●	
Compressions Stopped	○ ● ○ ○	

\*CPM = Compressions per minute  
 = Flashing